


Rhythm Multi LAL/AP Mattress Quick Reference Guide

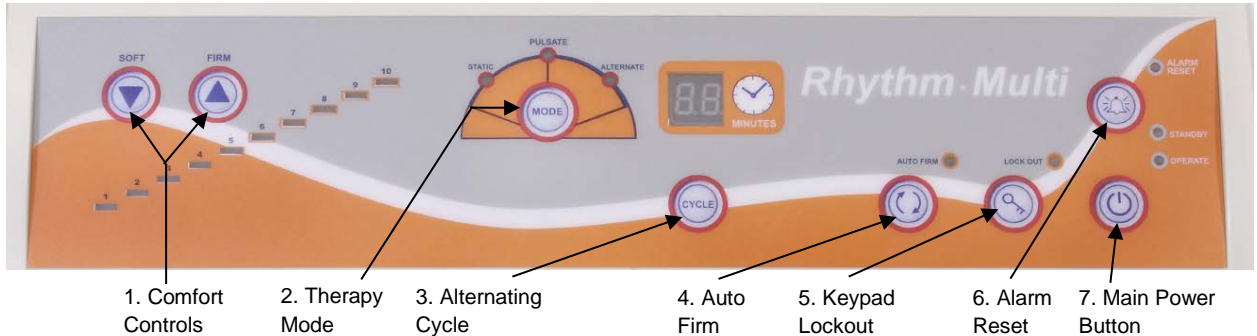

1. Comfort Controls

2. Therapy Mode

3. Alternating Cycle

4. Auto Firm

5. Keypad Lockout

6. Alarm Reset

7. Main Power Button

Explanation of the Rhythm Multi controls

1. **Comfort Controls** - Choose from 1 of 10 selectable levels of air pressure for support of the patient. Manufacturer recommends using a hand check to select proper pressure setting for patient.

2. **Therapy Mode** – Pressing the MODE button will toggle between static low air loss (STATIC), alternating pressure and pulsation therapies

3. **Alternating Cycle** – When the Therapy Mode is in the ALTERNATE setting, the CYCLE button provides a selectable alternating pressure cycle between 3 and 95 minutes

4. **Auto Firm** –Firms up the mattress to allow for easier egress, ingress or patient transfer.

5. **Keypad Lockout** - Press and hold for 5 seconds to lock/unlock all controls from patients or other non-caregivers

6. **Alarm Reset** – If the power failure alarm is triggered, this button will silence and reset the alarm.

7. **Main Power Button** – after the main power switch is turned on (side of pump) press and hold this button to move pump from Standby Mode to Operate Mode which activates the airflow.

Rhythm Multi LAL/AP Operational Guidelines

1. Carefully plug power cord of Control Unit into a properly grounded power source. Turn on the master mechanical power switch on the right side panel. The STANDBY LED will illuminate on the keypad.
2. Press the Main Power Button. The OPERATE LED will illuminate and the airflow will commence.
3. The Rhythm Multi mattress utilizes true low air loss therapy (1275L per minute) at all times during operation. Set optimal air pressure for the patient by using a hand check method.
4. The mattress defaults in STATIC mode. Press the MODE button once to change the therapy to PULSATE. This setting supplies pulsation therapy where the pressure in the mattress lowers for 90 seconds and then rises back to the original pressure for 90 seconds, a wave-like motion.
5. Press the MODE button a second time for the ALTERNATE mode. This setting provides 3 in 1 alternating pressure therapy for treatment and prevention of pressure ulcers. The cells inflate and deflate in a 3:1 cycle, meaning 2/3rds of the body is always supported by air at any one time.
6. The CYCLE button controls the alternating pressure cycle time in the ALTERNATE mode. CYCLE is selectable from 3 to 95 minutes. A CYCLE marks the time each 1/3 set of bladders are deflated.
7. PLEASE REMEMBER TO CLEAN THE FILTER ON THE BACK OF THE RHYTHM MULTI AT LEAST EVERY 30 DAYS. The black filter that is housed in the back of the Control Unit can be rinsed under running water and set to dry for 15 minutes. If you notice any dust clogs on the filter opening, use a vacuum to remove.