


Rhythm Turn Quick Reference Guide

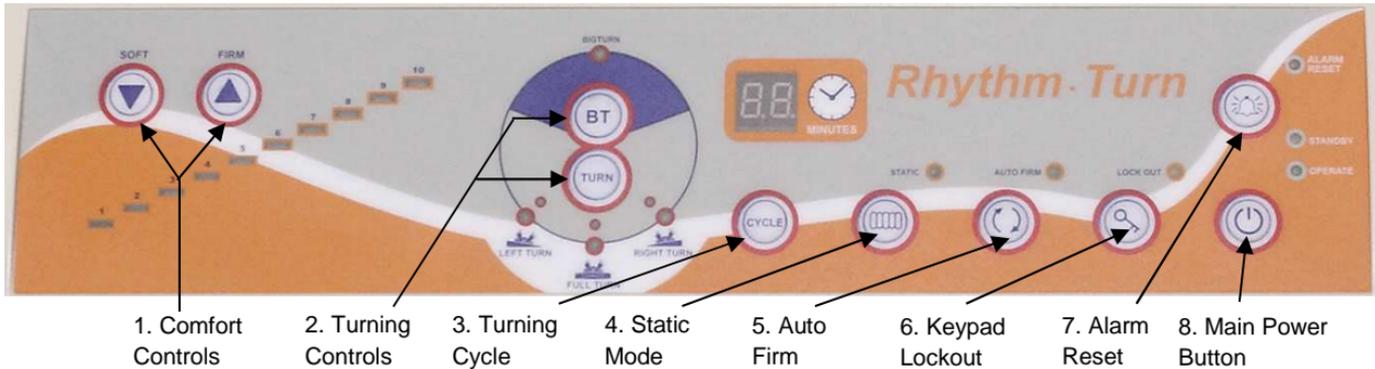

Explanation of the Rhythm Turn controls

<p>1. Comfort Controls - Choose from 1 of 10 selectable levels of air pressure for support of the patient. Manufacturer recommends using a hand check to select proper pressure setting for patient.</p>	<p>5. Auto Firm –Firms up the mattress to allow for easier egress, ingress or patient transfer.</p>
<p>2. Turning Controls – Selecting the “Turn” button provides for lateral rotation of 20 degrees. The “BT” button supplies a 40 degree turn.</p>	<p>6. Keypad Lockout - Press and hold for 5 seconds to lock/unlock all controls from patients or other non-caregivers</p>
<p>3. Turning Cycle – Allows for turning cycles of 3 to 95 minutes.</p>	<p>7. Alarm Reset – If the power failure alarm is triggered, this button will silence and reset the alarm.</p>
<p>4. Static Mode – When on static mode the mattress will deliver low air loss therapy only, no lateral rotation.</p>	<p>8. Main Power Button – after the main power switch is turned on (side of pump) press and hold this button to move pump from Standby Mode to Operate Mode which activates the airflow.</p>

Rhythm Turn Operational Guidelines

1. ****IMPORTANT - The side rails on the bed frame should be in the raised position while performing lateral rotation.**
2. ****IMPORTANT -You must hold down the black button on the top of the mattress air tube connector while attaching it to the port on the left side of the Control Unit. Failure to depress the button could result in damage to the metal attachment hook on the air tube connector.**
3. Carefully plug power cord of Control Unit into a properly grounded power source. Turn on the master mechanical power switch on the right side panel. The STANDBY LED will illuminate on the keypad.
4. Press the Main Power Button. The OPERATE LED will illuminate and the airflow will commence.
5. The Rhythm Turn mattress utilizes true low air loss therapy at all times during operation. Set optimal air pressure for the pressure by using a hand check method.
6. The mattress defaults in STATIC mode. Press the TURN button to activate 20 degrees lateral rotation and the desired position. LEFT TURN allows the mattress to turn to left and back to horizontal. RIGHT TURN would have the same effect but turning to the right. The FULL TURN allows for full function of turning to left and right and should always activate with timer setting. The timer can be set by "CYCLE" button.
7. The CYCLE may be set from 3 to 95 minutes. A CYCLE marks the time for each phase of the turn. For example, a 5 minute CYCLE on FULL TURN would consist of a 5 minute Right Turn, 5 minute horizontal period and a 5 minute Left Turn, and continue to cycle in that manner until the TURN function is turned off.
8. A CYCLE can also be set for "Hold" (shows Hd on the LED readout) in the LEFT TURN or RIGHT TURN setting. It allows a turn and indefinite hold for the patient.
9. PLEASE REMEMBER TO CLEAN THE FILTER ON THE BACK OF THE RHYTHM TURN AT LEAST EVERY 30 DAYS. The black filter that is housed in the back of the Control Unit can be rinsed under running water and set to dry for 15 minutes. If you notice any dust clogs on the filter opening, use a vacuum to remove.
10. To deflate the side bolsters for easier egress/ingress, find the zipper at the foot end in the middle of the bottom cover. Unzip and detach the quick connect on the red valve. To inflate, reattach the quick connect.