Blake Medical Distribution

Care and Cleaning of your G1, G2 and G3 Mattress
Soin et Le Nettoyage De Votre Matelas G1, G2 et G3

Blake

Medical Distribution
Ph#: 1-866-662-3459
Fax#: 1-905-560-1342

38 Lincoln St., Hamilton, ON L8L 7L5
www.blakemedical.ca


[Cleaning Your G1/G2/G3 Mattress]

The cleaning properties of the Geo-Matrix Mattress are unique. The cover is incontinent proof, fire resistant and machine washable. Machine drying is not recommended. The polymer gel is also washable, but as it is bonded to the foam layer, it cannot be machined washed. Instead, the gel layer can be cleaned with a damp cloth and light detergent. Note: Be sure to keep the foam layers away from any liquids. Also, avoid drying the mattress in direct sunlight. DO NOT USE BLEACH. Virox and other cleaning products without bleach are recommended.

Geo-Matrix Mattresses

Geo-Matrix Mattresses all have a foam base with inserts of gel. To clean the gel inserts, spray a mild detergent and wipe clean with a damp cloth. The mattress can then be turned upside down to air dry.

The mattresses are covered with the incontinent cover which can be machine washed and hanged to dry. If cover becomes excessively soiled, replace with a new cover.

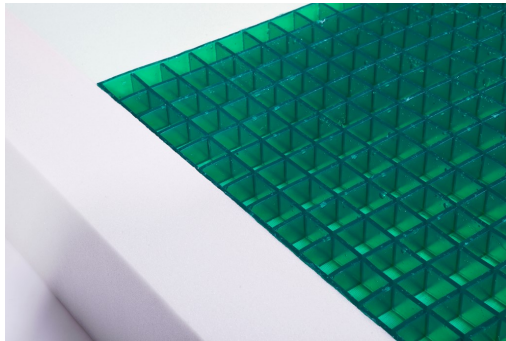[Care of your G1/G2/G3 Mattress]

The mattresses are designed to be used at room temperature. If it is subjected to extreme heat or cold, allow the mattress to acclimate to room temperature before using.

The gel and foam are light sensitive and will begin to breakdown if exposed for prolonged periods to direct sunlight. Avoid air-drying outside and exposure of the mattress to direct sunlight.

The mattresses are designed to be covered with fire sock as well as an incontinent cover. To ensure that the mattress retains its fire re-retardant characteristics, ensure that during reassembly both covers are in tact.

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