

Installation Instructions

Headrest



Blake
Medical Distribution

1-866-662-3459
www.blakemedical.ca



Thank you for choosing a Blake Medical headrest. The only headrest with Geo-Matrix gel...

Installation Tools Required: 10 mm Wrench and 4 mm Allen Key

Our headrests come with the following parts to set up your headrest.



1. Two Stud Plates with Four Nuts

2. Mounting Bracket Handle

3. Mounting Bracket

4. Mounting Bracket Clamp

The following steps below will provide you with the information needed to integrate this headrest to your chosen wheelchair backrest. If you are installing onto a Blake Medical Silverback backrest, please start at step four.

1. Supplied are two 3" **stud plates**. Insert these **stud plates** into the shell so that the threaded bolts are protruding through the external back of the shell. The flat portion of the **stud plates** should be flush with the shell.
2. Insert the **mounting bracket** into the four threaded bolts from the **stud plates**. The **mounting bracket** is the unit that fastens the multi axle hardware to the shell.
3. Using the provided four 10mm **nuts** to fasten the **mounting bracket** to the threaded bolts of the **stud plate**.

4. Slide the **mounting bracket clamp** into the **mounting bracket** and fasten the **mounting bracket handle** through the mounting bracket clamp and mounting bracket.
5. Leave **mounting bracket handle** loose enough to allow the vertical post of the headrest to slide into position.
6. To tighten the **mounting bracket**, turn the handle clockwise to lock the vertical post in place.
7. For headrest purchased follow instructions to mount headrest horizontal post onto the vertical post.
 - a. **Toolless Headrest** turn the **blue** adjustment handle counter clockwise to loosen the **black** knuckle on the vertical post to slide the horizontal post in place.
 - b. **Allen Key Headrest** use the Allen key counter clockwise to loosen the two bolts on the **black** knuckle on the vertical post to slide the horizontal post in place.
 - c. Once the headrest horizontal post is in the correct position use either **blue** adjustment handle or Allen key clockwise to tighten post into place.
8. Once both posts are in place you may adjust the headrest pad with the 4mm Allen key by turning the three bolts on the back of the pad counter clockwise. Once the pad is in place turn the bolts clockwise to lock the pad into position so that the user is comfortable and supported.



