Care & Cleaning

Geo-Matrix™ Cushion

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Votre Coussin Geo-Matrix^{MC}

Blake
Medical Distribution

1-866-662-3459
www.blakemedical.ca

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For daily cleaning wipe down the cover with soap and water. If fluids do soil the inner layers of the Geo-Matrix™ cushion, follow the instructions below.

Covers

Remove all contamination with cloth or paper towel, then wipe the surface with any of the following. 3M Hard Surface Disinfectant Cleaner Wipes, Wood Wyant ED Disinfectant, Virox 5, Oxvir TB, Virex® II 256 One-step Disinfectant Cleaner & Deodorant, Virex II 128 One-step Disinfectant Cleaner, Sanikleen and Super Sani Cloth Germicidal Disposable Wipe. Next disinfect liberally with any of the above wipes (avoid pooling of the disinfectant), wipe dry after 1-3 minutes. Next wipe the surface with a damp cloth and mild soap and finally a damp cloth with liberal amounts of water. Both covers are machine washable, up to 60 degrees C. Do not wring, lay flat to dry.

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Gel Sheets

The Geo-Matrix™ gel can go into a front load washing machine with no agitator a single gel sheet at a time. If your gel cushion is bonded it can not go into the washing machine. Use liquid detergent only with no bleach in any temperature water. If hand washing spray the gel with a mild non-abrasive detergent and wipe clean. In-house, Blake Medical uses “Urine Off”, “Virox” and “Sanicare” to disinfect Geo-Matrix™ product. Let the gel layers air dry and avoid drying in direct sunlight. Please note: do not use bleach to clean any Geo-Matrix™ product.

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Gel / Foam

To clean the Geo-Matrix™ gel, spray it with a mild non-abrasive detergent and wipe clean. In-house, Blake Medical uses “Urine Off”, “Virox” and “Sanicare” to disinfect Geo-Matrix™ product. Feel free to contact us with any questions about a disinfectant you may have prior to its use. Please note: do not use bleach to clean any Geo-Matrix™ product. **DO NOT WET FOAM.** Then the cushion can be turned upside down to air dry. Once dry put inner and outer cover back on.

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Care of your Geo-Matrix™ Cushion

The cushion is designed to be used at room temperature. If it is subjected to extreme heat or cold, allow the cushion to acclimate to room temperature before using. The gel layers are light sensitive and will begin to breakdown if exposed for prolonged periods to direct sunlight. Avoid air-drying outside and exposure of the gel layers to direct sunlight. The gel layers are designed to be covered with an inner and outer cover, ensure that during reassembly both covers are used.

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Reassembling your Geo-Matrix™ cushion after cleaning

Each cushion cover contains a patch that indicates the correct placement and orientation of the gel layers.

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