Cleaning Instructions

Geo-Matrix™ Mattress

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Matelas Geo-Matrix^{MC}

Blake
Medical Distribution

1-866-662-3459
www.blakemedical.ca

G Series Mattress Cleaning Instructions

Cleaning your Geo-Matrix™ Mattress

Geo-Matrix™ mattresses all have a foam base with inserts of gel. Any mattress with exposed gel can be cleaned, all other gel that has a fabric layer adhered to it can not be wiped. Do not wet this fabric. To clean the exposed gel inserts, spray it with a mild non-abrasive detergent and wipe clean with a damp cloth. In-house, Blake Medical uses “Urine Off”, “Virox” and “Sanicare” to disinfect Geo-Matrix™ product. Feel free to contact us with any questions about a disinfectant you may have prior to its use. Please note: do not use bleach to clean any Geo-Matrix™ product. **DO NOT WET FOAM.** The mattress can then be turned upside down to air dry. Avoid drying mattress in direct sunlight.

Cleaning your Mattress Cover

The mattresses are covered with an incontinent wipeable fabric. To clean remove all contamination with cloth or paper towel, then wipe the surface with any of the following. 3M Hard Surface Disinfectant Cleaner Wipes, Wood Wyant ED Disinfectant, Virox 5, Oxvir TB, Virex @ II 256 One-step Disinfectant Cleaner & Deodorant, Virex II 128 One-step Disinfectant Cleaner, Sanikleen and Super Sani Cloth Germicidal Disposable Wipe. Next disinfect liberally with any of the above wipes (avoid pooling of the disinfectant), wipe dry after 1-3 minutes. Next wipe the surface with a damp cloth and mild soap and finally a damp cloth with liberal amounts of water. The cover is also machine washable up to 60 degrees C and should be hanged to dry. **DO NOT USE BLEACH** when cleaning the cover, the G3 mattress cover fabric is the only one that supports the use of bleach and can be used on this cover only.

Maximum Washing Temperature: 60 degrees

If cover becomes excessively soiled, replace with a new cover.

Care of your Geo-Matrix™ Mattress

The mattresses are designed to be used at room temperature. If it is subjected to extreme heat or cold, allow the mattress time to acclimate to room temperature before using. The gel and foam are light sensitive and will begin to breakdown if exposed for prolonged periods to direct sunlight. Avoid air-drying outside and exposure of the mattress to direct sunlight. *(Images on back)*

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