Cleaning Instructions

Gel Overlay

Instructions de nettoyage du

Superposition de gel

Blake
Medical Distribution

1-866-662-3459
www.blakemedical.ca

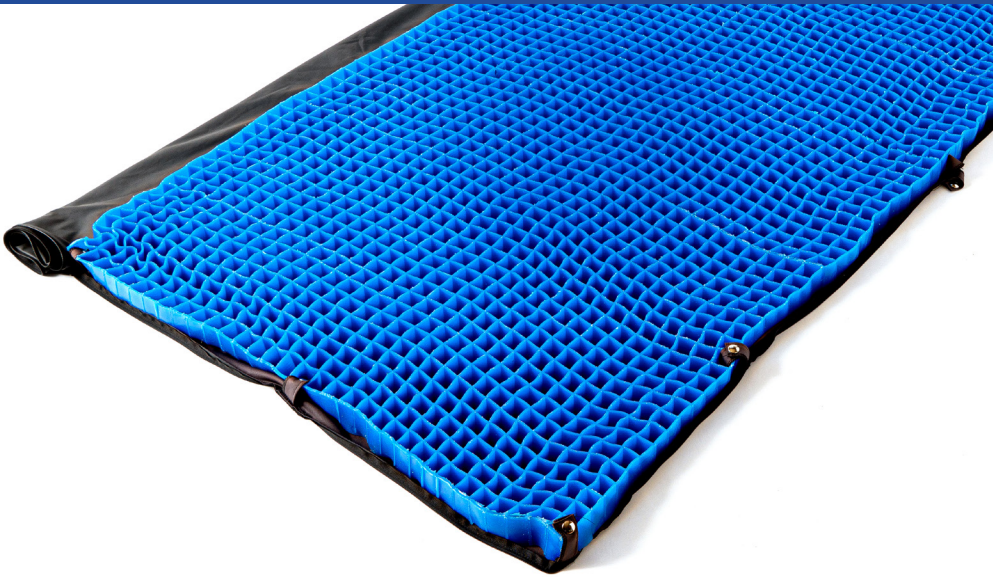
For daily cleanings all that is needed is a wipe down of the cover. This cover is a fluid proof fabric and will prevent most fluids from permeating through to the overlay. For these daily cleanings soap and water will be all that is needed. If fluids do soil the overlay follow the instructions below.

Covers

Remove all contamination with cloth or paper towel, then wipe the surface with any of the following. 3M Hard Surface Disinfectant Cleaner Wipes, Wood Wyant ED Disinfectant, Virox 5, Oxvir TB, Virex ® II 256 One-step Disinfectant Cleaner & Deodorant, Virex II 128 One-step Disinfectant Cleaner, Sanikleen and Super Sani Cloth Germicidal Disposable Wipe. Next disinfect liberally with any of the above wipes (avoid pooling of the disinfectant), wipe dry after 1-3 minutes. Next wipe the surface with a damp cloth and mild soap and finally a damp cloth with liberal amounts of water. The cover is machine washable, up to 60 degrees C. Do not wring, lay flat to dry.

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Gel Overlay Cleaning

The Geo-Matrix™ gel cannot go into the washing machine as it is too heavy and could damage your machine. To clean the Geo-Matrix™ gel, spray it with a mild non-abrasive detergent and wipe clean. In-house, Blake Medical uses “Urine Off”, “Virox” and “Sanicare” to disinfect Geo-Matrix™ product. Feel free to contact us with any questions about a disinfectant you may have prior to its use.

Let the gel air dry with the cells facing down and avoid drying in direct sunlight.

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Reassembling your Geo-Matrix™ overlay after cleaning

Take the cover and lay the bottom of it on a flat surface. Place the Geo-Matrix™ gel onto the cover with the open end of the columns facing up. Once the gel is positioned centered in the cover, take the snap loops and put one through the designated hole cut in the gel and the other around the outside of the gel and snap them together to keep the gel in place. Do this for all snap loops on the cover. Once complete you can flip the top fabric of the cover over the gel and zip up.

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Care of your Overlay

Blake Medical's Overlays are designed to be used at room temperature. If it is subjected to extreme heat or cold, allow the overlay to acclimate to room temperature before using.

Geo-Matrix™ gel is UV sensitive and will begin to breakdown if exposed for prolonged periods of direct sunlight, avoid air-drying outside.

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