



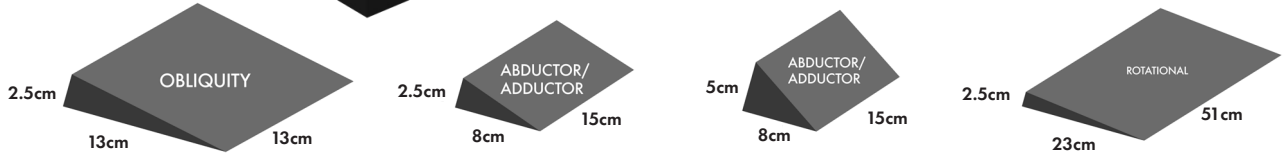

Proudly made
and manufactured
in Canada!

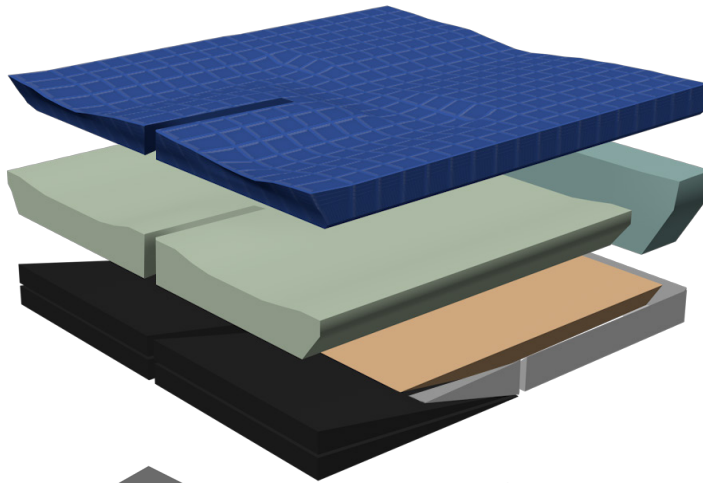

Product Specifications

CONSTRUCTION

Closed cell foam base, dual high-density foam core with a firm sacral shelf and cascading Pillow Top gel topper

COVERS

Two outer cover options: Breathable Spacetex fabric with incontinent protection OR Elastatex soft touch outer incontinent cover. Inner cover: Elastatex breathable, wipe-able, antibacterial, fluid proof cover.

STANDARD SIZES

35x35, 35x38, 35x40, 35x43, 35x48, 35x50, 38x35, 38x38, 38x40, 38x43, 38x45, 38x48, 38x50, 40x35, 40x38, 40x40, 40x43, 40x45, 40x48, 40x50, 40x53, 40x55, 43x35, 43x38, 43x40, 43x43, 43x45, 43x48, 43x50, 43x55, 45x30, 45x33, 54x35, 45x38, 45x40, 45x43, 45x45, 45x48, 45x50, 45x53, 45x55, 48x35, 48x38, 48x40, 48x43, 48x45, 48x48, 48x50, 48x53, 48x55, 50x35, 50x38, 50x40, 50x43, 50x45, 50x48, 50x50, 50x53, 50x55, 53x35, 53x38, 53x40, 53x43, 53x45, 53x48, 53x50, 53x53, 53x55, 55x35, 55x38, 55x40, 55x43, 55x45, 55x48, 55x50, 55x53, 55x55, 58x40, 58x43, 58x45, 58x48, 58x50, 58x53, 58x55, 60x40, 60x43, 60x45, 60x48, 60x50, 60x53, 60x53, 60x55, 60x55

CUSTOM SIZES

All sizes available

PRODUCT CODE


GSP"WxD" eg. GSP4545

WEIGHT CAPACITY


125kg (Cushions 45x45cm and above have a weight capacity of 135 kg)

The Geo-Matrix™ Split cushion is often used to provide comfort for users who require gentle positioning and contouring with excellent pressure redistribution and vibration control. The Split cushion features an 8" split directly through the centre of the pommel. This allows wedges to be inserted under one or even both sides of the cushion's leg trough region to accommodate the desired hip angle required for each individual user. The Split cushion comes with a "fit kit" of various wedges. Each wedge secures to the base of the cushion using velcro. These wedges can be added or removed as needed for on the spot customization. This also allows for modifications to be easily made to the cushion if the user's condition should change.

To further the Splits custom feel and protection there are different insert options that may be added underneath the high risk coccyx section. These inserts are:

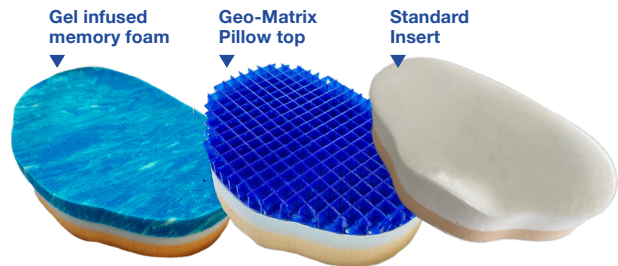

Blake Medical esteemed distributor:

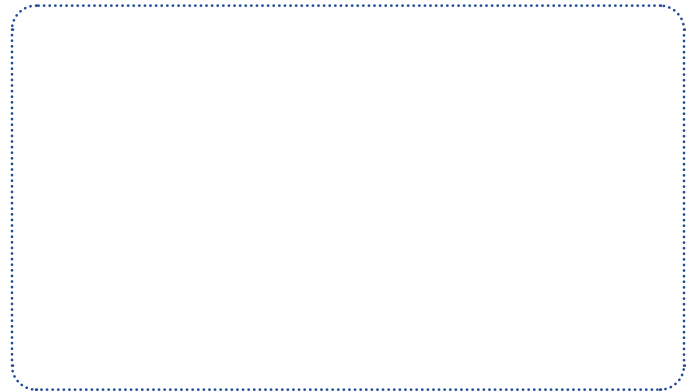

GEL & FOAM CUSHION Geo-Matrix Split




WASHING INSTRUCTIONS

Covers: 60°C in washing machine. Hang to air dry. Do not bleach.

Geo-Matrix™ gel and foam: Wipe down gel using any cleaner that does not contain bleach and allow to air dry.

For more specific cleaning instructions please visit our website or user's manual.

All of Blake Medical products are free from phthalates and latex. This includes all medical fabrics, Geo-Matrix™ gel, moulded polyurethane, standard polyurethane and specialty foam. All cushions meet EN-1021 part 1 & 2 requirements for fire retardancy.

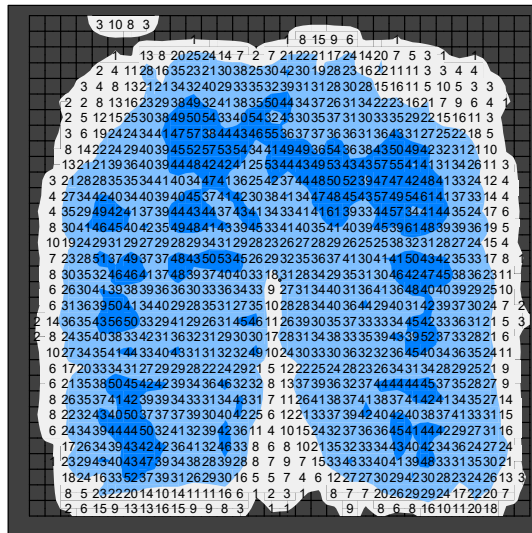

ADDITIONAL BENEFITS

- Cushion remains 2-3 degrees cooler under seated body temperature to assist with microclimate management.
- Minimizes shear through column segmentation.
- Geo-Matrix™ Gel does not change structure in climates of extreme heat nor extreme cold.
- Geo-Matrix™ gel is recyclable through Blake Medical.

Client: Male 71kg

Advantages of Geo-Matrix™ Gel

